


KaVo Maintenance Guide

KaVo Multiflex High Speed Handpieces

Attention: To insure warranty through proper maintenance with KaVo Spray, follow the steps exactly as shown.



Remove the Multiflex handpiece from the Multiflex coupler.

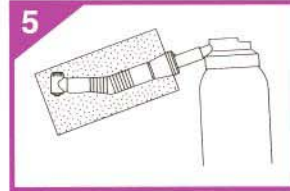


Fit the handpiece on the KaVo Spray with the Multiflex Nozzle. Cover with KaVo Clean Pac or paper towel and spray through for 1 second.

Other maintenance fluids are not recommended. If a large amount of debris appears, repeat the process.



Remove bur.

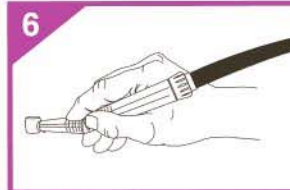

Fit the handpiece without bur on the Multiflex coupler or air station, allow it to run for 20 seconds and wipe off excess lubricant.



Scrub handpiece under cold running water with a soft toothbrush.



Sterilize Multiflex High Speed handpieces without bur up to maximum 135°C (275°F) in an autoclave or chemiclave.




Do not place or immerse in disinfectant! Do not combine disinfecting with sterilization.



The Multiflex coupler cannot be sterilized.


- Condensation may occur after sterilization. Please keep handpieces in an upright position when storing.
- KaVo Multiflex High Speed Handpieces must be lubricated before every sterilization cycle with KaVo Spray.
- Do not spray after sterilization.
- If alcohol wipes are used instead of sterilization, lubricate with KaVo Spray after disinfecting.

Multiflex Nozzle

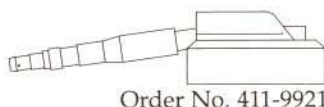

Order No. 411-9921

Quick Nozzle

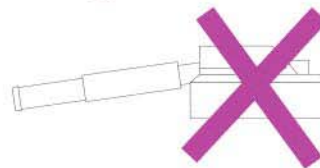

KaVo. Dental Excellence.